

scil Technical Support

scil Vet ABC Plus Blockage Removal

1. Log in as ADMIN (Password:SCIL)

2. Select the Maintenance menu



3. Select Serv. Tab at the top

4. Select Customer Service Menu



5. Select Chambers Draining and Press Start Cycle Icon



6. Open Front Door

7. Put 3mL of 50/50 Bleach/Distilled water mixture in each of the chambers

8. Close door

9. Select Backflush and Press the Start Cycle Icon to run the Backflush



10. Repeat the Backflush 10 times

11. Select Drain Chambers and Press the Start Cycle Icon



12. Press the Status Icon at the top right of the screen



13. Press the Start Up Button



14. Run a Quality Control

15. Fax Results to 1-705-726-9202 if it does not pass. Include your clinic name and your name on the fax as well please.