



## **SCIL Vet ABC**

### **Chamber Blockage Removal**

Press 4 - Service

Press 2 - Drain Chambers

3mL 50/50 Distilled water/Bleach in RBC Chamber

3mL 50/50 Distilled water/Bleach in Mixing Chamber

Press 5 – Transfer

Press 1 – Back Flush 10 to 15 times

Press 2 – Drain Chambers

Escape back to Main Menu

Press the Start Up Button

Run a Quality Control if you are trying to resolve a problem

If the problem is not resolved, repeat the procedure