

Extensive Cleaning Procedure

1. Turn on scil Vet ABC / scil Vet ABC Plus⁺ and abort warm up and start up cycle by hitting escape
2. Fill a sample tube with 1 ml of Minoclair (50% bleach/50% Distilled Water).
3. Run a dog measurement and use the Minoclair filled sample tube as your sample.
4. Once you press the sample bar to aspirate and the needle has moved up, immediately turn off the analyzer by using the power switch at the back
5. Wait for 20 minutes.
6. Turn on scil Vet ABC / scil Vet ABC Plus⁺ and abort the warm up and start up cycle by pressing the escape button (make sure that the analyzer does not begin to prime or the Minoclair will be primed out of the needle).
7. Perform a blockage removal

