
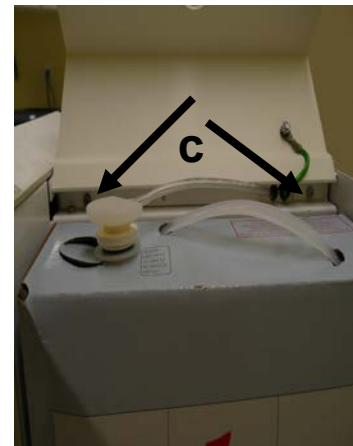


# scil Technical Support


## HOW TO REATTACH A TUBE

1. Open the front door of the Vet abc.
2. Remove the cover of the Vet abc.
  - a) Remove the two screws on the back of the instrument at the far right top and bottom.
  - b) Remove the screw on the side under the door latch.
  - c) Loosen the two screws above the reagent pack. Do not remove.
  - d) Lift off the instrument cover.

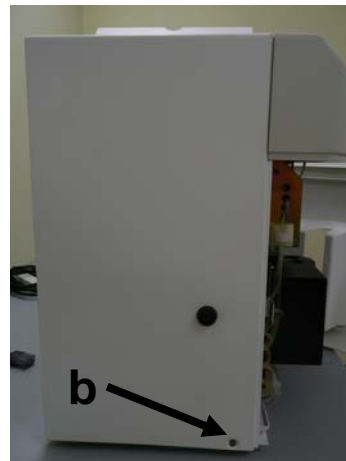

3. Inspect the valves on the right hand side of the machine (your left). Look very closely for a tube that has popped off. It may be sitting very close to the valve.

## scil Technical Support

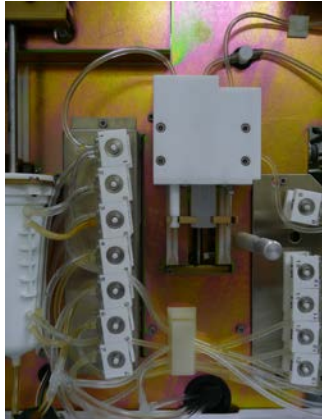

4. Once you have found the tube that has popped off carefully cut the very end of it off. (Approx. 5mm)
5. Replace it back on the nub of the valve firmly.
6. Run a concentrated cleaning.
7. Run a start up cycle.
8. Run a blood control. If your results are not in range please fax them to Vet Novations at 705-726-9202.