

## SCIL VET ABC

### Mixing Chamber Cleaning

1. Open the front door of the Vet ABC
2. If there is a black cover over your mixing and RBC chamber you must remove it by loosening the top and bottom screws and sliding the cover upwards, then pulling it off towards you.



Screw # 2 is located inside the black box. You will need to put the Vet Abc on the edge of a table and bend down to see it through the opening that the tubes come out of.

3. Pull the mixing chamber out of the clip. And disconnect the tube from the bottom of the chamber. Clean the chamber under water in a sink. Make sure there are no stained areas or crystals around the rim of the chamber.



4. Reattach the Mixing Chamber to the tubing and push the mixing chamber back into the clip.
5. Put the black cover back on. Be sure not to pinch any tubing. It is very easy to do. Feed the tubing through the bottom of the cover first and then slide the cover over the chambers. All the tubing should move freely through the opening in the bottom. Check the left side of the cover as well for any pinched tubing.