

Eurolyser solo

Warm reagents for 10 minutes

The analyzer has been calibrated based on a 10 minute warming period of the reagents. Results may not be accurate if the warming period of 10 minutes is not followed.

Colour of ERS Cap	Test	Sample Material	Sample Volume	Measuring Range	Species	
violet	T4	Serum/ Lithium-Heparin Plasma	20 µl	20 µl sample: 0 - 8 µg/dl 0 - 104 nmol/l	5 µl sample: 6 - 32 µg/dl 78 - 416 nmol/l	all species
green	Fructosamine	EDTA-Plasma, Lithium-Heparin Plasma, Serum	20 µl	0 - 2600 µmol/l		all species
yellow	Fibrinogen	Lithium-Heparin Plasma, Citrate Plasma	5 µl	0 - 800 mg/dl 0 - 8 g/l		horse
blue	cCRP	Serum/Plasma	5 µl	10 - 200 mg/l		dog
red	SAA	Serum	5 µl	10 - 500 µg/ml 10 - 500 mg /l		cat, horse
white	Ammonia (NH ₃)	EDTA-Plasma	20 µl	10 - 1100 µmol/l		dog, cat
orange	Lipase (Pancreas-specific)	Serum/ Lithium-Heparin Plasma	20 µl	20 µl sample: 25 - 300 U/l	5 µl sample: 250 - 1200 U/l	dog, cat
white	GLDH	Serum/ Lithium-Heparin Plasma	100 µl	6-120 U/l		all species
red	Phenobarbital	Serum	20 µl	10-100 µg/ml		dog, cat, horse

References:

Manufacturer´s Data: Eurolyser Diagnostica GmbH



scil animal care company
341 King St, Unit 1
Barrie, ON L4N 6B5
Tel: 866-382-6937
Fax: 705-726-9202
Email: infoca@scilvet.com
www.scilvet.com