

# Technique Chart for the scil SXR-HF with Canon DR

| AREA/VIEW                      | MA  | Time  | 2  | 3  | 4  | 5  | 6  | 7  | 8  | 9  | 10 | 11 | 12 | 13 | 14 | 15 |
|--------------------------------|-----|-------|----|----|----|----|----|----|----|----|----|----|----|----|----|----|
| <b>BUCKY</b>                   |     |       |    |    |    |    |    |    |    |    |    |    |    |    |    |    |
| <b>ABDOMEN</b>                 | 300 | .1/60 | 56 | 58 | 60 | 62 | 64 | 66 | 68 | 70 | 72 | 74 | 76 | 78 | 80 | 82 |
| <b>CANINE THORAX</b>           | 300 | .1/40 |    |    |    |    |    |    |    | 60 | 62 | 64 | 66 | 68 | 70 | 72 |
| <b>SPINE/HEAD &amp; PELVIS</b> | 300 | .1/20 | 44 | 46 | 48 | 50 | 52 | 54 | 56 | 58 | 60 | 62 | 64 | 66 | 69 | 73 |
| <b>EXTREMITIES</b>             | 300 | .1/60 | 43 | 45 | 48 | 50 | 42 | 54 | 57 | 60 | 62 | 64 | 66 | 68 | 70 | 72 |
| <b>FELINE THORAX</b>           | 100 | .1/40 | 46 | 48 | 50 | 52 | 54 | 56 | 58 | 60 | 62 | 64 | 66 | 68 | 70 | 72 |

| AREA/VIEW                      | MA  | Time  | 16 | 17 | 18 | 19 | 20 | 21 | 22 | 23  | 24  | 25  | 26  | 27  | 28  | 29  |
|--------------------------------|-----|-------|----|----|----|----|----|----|----|-----|-----|-----|-----|-----|-----|-----|
| <b>BUCKY</b>                   |     |       |    |    |    |    |    |    |    |     |     |     |     |     |     |     |
| <b>ABDOMEN</b>                 | 300 | .1/60 | 84 | 86 | 88 | 90 | 92 | 95 | 98 | 101 | 104 | 107 | 110 | 113 | 116 | 120 |
| <b>CANINE THORAX</b>           | 300 | .1/40 | 74 | 76 | 78 | 80 | 82 | 84 | 86 | 88  | 90  | 92  | 94  | 96  | 98  | 100 |
| <b>SPINE/HEAD &amp; PELVIS</b> | 300 | .1/20 | 77 | 80 | 83 | 88 | 91 | 94 | 97 | 100 | 102 | 104 | 106 | 108 | 110 | 112 |
| <b>EXTREMITIES</b>             | 300 | .1/60 | 74 | 76 | 78 | 80 | 82 | 84 | 86 | 88  |     |     |     |     |     |     |
| <b>FELINE THORAX</b>           | 100 | .1/40 | 74 | 76 | 78 | 80 |    |    |    |     |     |     |     |     |     |     |

